



## Cleaning NuStep Exercise Equipment

### Supplies:

- EPA registered quaternary ammonium disinfectant wipes, e.g., Super Sani-Cloth® or other disinfectant wipes with similar formulations.

— OR —

- Mild, non-toxic and non-abrasive spray cleaner
- Soft cloth



Do not use bleach or hydrogen peroxide-based cleaners. They may discolor or degrade certain contact surfaces over time. Do not use cleaners or disinfectants with concentrations of isopropyl alcohol greater than 70%.

### Frequency:

For best results, clean after every use.

### Directions:

1. Dispense disinfectant wipe from container. Follow manufacturer guidelines for use and disposal as shown [here](#) (for reference).  
— OR — Spray cleaner onto cloth (do not spray directly on display screen)
2. Wipe all contact surface areas shown below.
3. Allow disinfected surfaces to air dry before NuStep use.

## NuStep Cross Trainers



*NuStep cross trainer features vary based on models. The T6 is shown above.*

## NuStep Upper Body Ergometer



## NuStep Recumbent Bike



For questions or more information, please contact NuStep at:  
800-322-2209 or [support@nustep.com](mailto:support@nustep.com)

PN 52506 Cleaning NuStep Exercise Equipment, Rev B



[nustep.com](http://nustep.com)